



BEDSIDE COMMODOE USER GUIDE

Safe Setup

- Place the commode on a **firm, level surface**.
- Ensure all legs are adjusted to the **same height** and securely locked.
- Confirm all push buttons and adjustment pins are fully engaged.
- If using the commode over a toilet, position it securely and verify stability before use.
- Do not use the commode if any parts are loose, damaged, or missing.

Before Each Use

- ✓ Verify the commode is stable and does not wobble.
- ✓ Ensure the bucket is securely in place (if used).
- ✓ Confirm armrests and frame are secure.
- ✓ Wear non-slip footwear when transferring.

Safe Use

- Follow your healthcare provider's instructions.
- Use the armrests to assist when sitting or standing.
- Transfer slowly and carefully.
- Do **not** stand on the commode.
- Do **not** exceed the maximum weight capacity specified by the manufacturer.
- Children should only use the commode with adult supervision.

Cleaning & Maintenance

- Empty the bucket after each use.
- Clean the bucket, seat, and frame with warm water and mild soap or an approved disinfectant.
- Dry all surfaces thoroughly before reuse.
- Inspect the commode regularly for:
 - Loose hardware
 - Cracks or damaged parts
 - Worn rubber tips
- Contact Prime Care Medical Equipment if repairs or replacement parts are needed.

Safety Precautions

- ⚠ Do not use on uneven or slippery surfaces.
- ⚠ Do not modify the commode.
- ⚠ Keep hands clear of moving parts during adjustments.
- ⚠ Replace worn or damaged parts immediately.
- ⚠ If you experience dizziness or difficulty transferring, ask a caregiver for assistance.