



MANUAL WHEELCHAIR USER GUIDE

BEFORE EACH USE

- ☐ Make sure the brakes lock securely.
- ☐ Check that tires are properly inflated (if pneumatic) and not damaged.
- ☐ Ensure footrests, armrests, and wheels are securely attached.
- ☐ Verify that the seat and back upholstery are not torn or loose.
- ☐ Remove loose objects from the seat and footrests.

SAFE OPERATION

- Always lock both wheel locks before getting in or out of the wheelchair.
- Never stand on the footrests.
- Keep both feet on the footrests while seated.
- Use caution on uneven surfaces, ramps, curbs, and inclines.
- Avoid steep hills or wet, icy, or slippery surfaces.
- Do not exceed the wheelchair's maximum weight capacity.
- Keep hands and fingers away from moving wheels and pinch points.
- Children should not operate the wheelchair without adult supervision.

TRANSFERS

1. Position the wheelchair as close as possible to the bed, chair, or toilet.
2. Lock both wheel locks.
3. Swing away or remove footrests if applicable.
4. Transfer slowly using approved techniques or assistance if needed.
5. Never use the footrests or armrests to support your full body weight unless designed for that purpose.

FOLDING & TRANSPORT

- Remove or swing away footrests if necessary.
- Lift the seat upholstery from the center to fold the wheelchair.
- Secure the wheelchair during vehicle transport.
- Unless the wheelchair is specifically approved as a transportation seat, transfer to the vehicle seat and wear the vehicle's seat belt.

CLEANING & MAINTENANCE

- Wipe the frame with mild soap and water.
- Dry thoroughly after cleaning.
- Inspect wheels, brakes, upholstery, and hardware regularly.
- Tighten loose hardware only if instructed by the manufacturer.
- Do not modify the wheelchair or install unauthorized accessories.