



PATIENT LIFT & SLING USER GUIDE

BEFORE EACH USE

- ✓ Inspect the lift, sling, hooks, straps, casters, and spreader bar for wear or damage.
- ✓ Verify the sling is the correct size and weight capacity for the patient.
- ✓ Confirm all sling straps are securely attached.
- ✓ Ensure the lift battery is charged (powered lifts).
- ✓ Do **NOT** use damaged equipment.

APPLYING THE SLING

- Position the sling according to manufacturer instructions.
- Smooth wrinkles and center the patient in the sling.
- Attach all loops evenly and verify they are securely connected.
- Never use a sling with torn fabric or damaged stitching.

SAFE TRANSFER

- Explain the procedure before beginning.
- Lock the bed or wheelchair before positioning the patient.
- Raise the patient slowly while checking sling placement.
- Keep the patient centered over the lift base.
- Move the lift slowly using the push handles only.
- Lower the patient carefully before removing the sling.
- Never leave a patient unattended while suspended.

SAFETY PRECAUTIONS

- ⚠ Never exceed the lift or sling weight capacity.
- ⚠ Use only manufacturer-approved slings.
- ⚠ Never stand or walk while suspended in the lift.
- ⚠ Keep hands and feet away from moving parts.
- ⚠ Use caution on ramps, thresholds, wet floors, and uneven surfaces.
- ⚠ Recharge powered lifts after use as recommended.
- ⚠ Report damaged or malfunctioning equipment immediately.

CLEANING & MAINTENANCE

- Clean the lift with a mild detergent or approved disinfectant.
- Launder the sling according to manufacturer instructions.
- Inspect straps, buckles, hooks, and wheels regularly.
- Keep batteries charged and report unusual noises or operating problems.